

Laerdal Suction Unit Quick Reference Guide

Caution

Extreme care must be taken to ensure that patient tubing is connected to the correct port.

Incorrect fitment will damage unit beyond repair

Suction Unit to Canister Tubing

Connect to grey attachment on canister

AC Power Indicator

If light not illuminated, confirm unit is correctly plugged in.
If light still does not illuminate, contact EBME on ext 3332

On/off switch

Switches unit on or off



Patient Suction Tubing Connection

Connect to white attachment on canister

Patient Tubing

Securely attached to side of unit

Daily check

1. Press and hold the "Test button while setting the suction dial to 500+mmhg
2. As soon as the second LED comes on fully block the end of the suction tube
3. Keep the tube occluded while the second, third and fourth LED comes on
4. Release the tube when the first LED comes on again